

POST THIS SIDE IN YOUR STERILIZATION ROOM

Enduro Repair

VECTOR

High-Performance Lubricant For High-Performing Handpieces

Never Lube the wrong spot!
Use the correct adapter!



- A KaVo .. VL-AK
- B Star VL-AT
- C NSK VL-AN
- D 4 Hole .. VL-AS
- E W&H VL-AW
- B E-Type VL-AE
- C Point ... VL-AU

The Ultimate Lubricant For Any Brand Of Handpiece



Use of this lubricant will not void factory warranty

1

Spray

Using the correct adapter, spray Vector Spray into the drive line or the hole indicated on the diagram above. Spray for approx. 2 seconds -

Count 1.... 2....



VL-AS Adapter



VL-AK Adapter

2

Run - 5 to 10 seconds

Run the handpiece to expel out excess lubricant. This should be done at the control unit in the operatory or at a purge station in the sterilization room

Count 1... 2... 3... 4 ... 5...



3

Wipe with alcohol to remove excess lubricant

Wipe down the exterior of the handpiece with a 2x2 lightly dampened with alcohol. Use pure rubbing alcohol only, many over-the-counter "alcohol wipes" have other solutions added to them, this can cause spotting of the exterior and **severe damage** to the internal parts of the handpiece.



4

Bag and Autoclave

Sterilize handpiece as per manufactures instructions. Always allow the handpieces to go through the "Dry Cycle" on your sterilizer. **Note:** We recommend using sterilization bags with one side paper and one side plastic.



5

Let Cool

Remove the handpiece from the sterilizer and let them cool to room temperature. Running "Hot" handpieces will severely reduce the life of the turbine. **Don't Re-Lubricate**



- With Vector Spray You Lubricate Only Once!

**CAUTION - * Never Wipe or Spray With Surface Disinfectants
* Never Submerge In Water * Never Place Handpieces Into the Ultrasonic**